Safety and Performance Information

English

TH02F, TH03F, TH09F, THD2FE, THAC2F, THi0F Non-contact Forehead Thermometer

● General

Intended use:

The non-contact forehead thermometer is electronic thermometer using an infrared detector (thermopile detector) to detect body temperature from forehead center in people of all ages.

Intended operator:

The intended operator should have at least eight years of education, including patients or other operators.

● Symbols

Symbol Descriptions					
	The CE mark and Notified Body Registration Numbers, the requirement of Annex II from Medical Device Directive 93/42/EEC are met.		Please do not dispose of the product in the household waste at the end of its useful life. Disposal can take place at appropriate collection points provided in your country.		Authorized representative in the European community
	Caution		Follow instructions for use		Manufacturer
	BF type applied part		To protect the environment, dispose of empty batteries at your retail store or at appropriate collection sites according to national or local regulations.		Paper Recycling
IP22	Classification for water ingress and particulate matter.		Stand-by (THD2FE, THi0F)		

● Safety Information

- ⚠ Choking from swallowing small parts and batteries by children or pets is possible, please keep small parts and batteries at places where children and pets can't reach.
- ⚠ The device should not submerge into any liquids and expose it to direct moisture, and please keep away from direct sunlight.
- ⚠ If there is certain temperature difference between the locations where the device is stored and where you are going to measure, subject and the device should stay in the same room for at least 15 minutes before measurement.
- ⚠ Always make sure the probe lens is clean without any damage.
- ⚠ Insert the batteries correctly, and the positive (+) side up and the negative (-) side pointed down. (TH02F, TH03F, TH09F, THi0F)
- ⚠ If the device is not to be used for a long time, turn the power off, remove and store the batteries in a cool and dry place. (THD2FE)
- ⚠ Turn the rotary cap back in place to prevent dirt and scratches from damaging the device. (THAC2F)
- ⚠ Store the thermometer with the cap in place to prevent dirt and scratches from damaging the device. (THi0F)
- ⚠ No modification of this equipment is allowed.

● Technical Information

- Forehead mode Temperature measurement range: 34~42.2°C (93.2~108°F) (TH02F, TH03F, THD2FE, THAC2F)
Forehead mode Temperature measurement range: 28~43°C (82.4~109.4°F) (TH09F)
- Forehead mode Temperature measurement range: 34~42.2°C (93.2~108°F); Surface mode Temperature measurement range: -22~80°C (-7.6~176°F) (THi0F)

- Operating temperature range: 10~40°C (50~104°F), 15%~85% RH
- Storage temperature range: It should be stored at room temperature between -20~+50°C, RH≤85%
Transportation temperature shall be less than 70°C, RH≤95%
- Atmospheric pressure: 800~1013 hPa
- Comply with ASTM E1965-98, EN ISO 80601-2-56, IEC/EN60601-1-2(EMC), IEC/EN60601-1(Safety) standards, ISO10993, RoHS.
- Forehead mode Accuracy: ±0.2 °C (0.4 °F) within 35~42°C (95~107.6°F), ±0.3 °C (0.5 °F) for other range.
- Temperature Awareness Signal °C / °F switch function.
- 25 sets memory (**TH03F, THD2FE, THAC2F, THi0F**); 1 sets memory (**TH02F, TH09F**)
- Blue LED Backlight & Mute mode (**THD2FE**)
- Battery: one lithium cell battery CR2032 *1pcs (**TH02F, TH03F, TH09F, THAC2F, THi0F**); AAA x 2 pcs (**THD2FE**)
- Battery life: around 3,000 continuous readings.
- Enclosure Rating: IP22
- Expected Service Life: 4 years

Copyright Notice

The entire contents of this manual are copyrighted by Radiant Innovation Inc. All rights are reserved.

Radiant Innovation Inc.
[Http://www.radiantek.com.tw](http://www.radiantek.com.tw)
 Add: 1F, No.3, Industrial East 9th Road,
 Science-Based Industrial Park, HsinChu, Taiwan 300.

Medical Technology Promedt Consulting GmbH
 Add: Ernst-Heckel-Straße 7, 66386 St. Ingbert, Germany

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L'intégralité du contenu de ce manuel est la propriété de Radiant Innovation Inc. Tous droits réservés.